N.E.W. *New Energy Waver*®

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Assembly instructions

The NEW ENERGY WAYER consists of 2
parts.

Remove both parts of the packaging.

Carefully remove the outer packaging and
attach the shorter part to the marked place
with the engine part.

Note the arrow on the gold Ring. Put both
parts together carefully and turn about 1/2
cm, depending on how you hold the device,
turn it to the right or left.

When assembling the NEW ENERGY
WAYER, please ensure, that the lower part
in the bayonet lock is properly engaged and
is firmly seated - otherwise the lower part
can become loose and when waving the
crystal tube may be damaged.

Now the device is ready for use.
Note the application suggestions of the
device.

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Safety instructions

1. Use the appropriate 220V power
connection
2. The device is equipped with a shock-sen-
sitive crystal tube
3. To avoid overheating, do not block airflow
4. To protect against electric shock, the
cord, plug and body of the device shall not
be allowed to come into contact with water
or other liquids.
5. Unplug the device when not in use.
6. Only use the device for intended
applications.
7. Use no more than 60 minutes - allow to
cool before reuse.

8. Overheat protection cuts off the power supply when the motor overheats and turns it on again automatically after the motor has cooled down.
9. Never let the cable, plug and device come into contact with water.
10. If the power cord is damaged, it must be replaced by a qualified person to avoid a hazard.
11. Never leave the device switched on unattended.
12. Do not switch on the device if it is defective.
13. Children are not allowed to operate this device without supervision.
14. Keep enough distance when applying to avoid burns.
15. Close your eyes when applying to this area.
16. Metallic implants, pacemakers, stents, drug pumps or fillers can heat up, skip this area.
17. Do not use if pregnant, breastfeeding or during monthly bleeding.
18. The device is a precision device, the user must not disassemble the device, otherwise the functionality can not be guaranteed.
19. Find out about the applications and handling of the NEW ENERGY WAYER at our workshops:
see: www.new-energy-waver.com

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One year guarantee (from purchase date with receipt)

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Please note the safety instructions for warranty guarantee!

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If the product is defective during the warranty period due to defects in material and workmanship, please contact DaHannes.

Die Garantie erlischt bei/ the following conditions are not within the warranty of the device:

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